



22 April 2022

To strengthen the bearing in rigidifying push bar on VIP 2

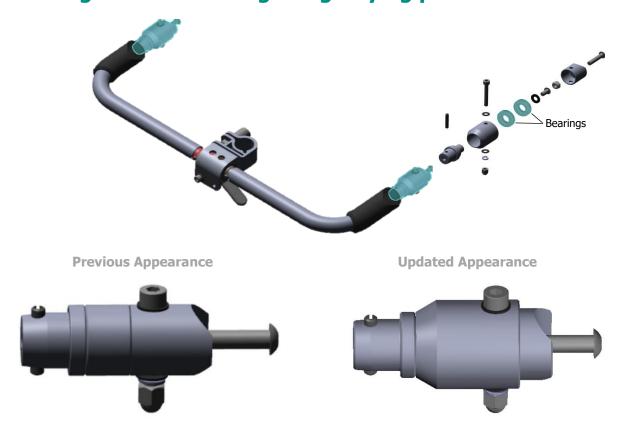

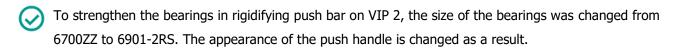

From now on, when you have to replace a previous version bearing (6700ZZ), we suggest your technicians change <u>both sides</u> directly to the updated version(6901-2RS) to increase its service life.

- The updated version was implemented to the VIP2 that started by the batch of Serial No. TM22-003955 on 2022 January 27th.
- The latest parts catalogue of VIP2, version 004, is already updated on Karma Website.
- The updated version is backward compatible with the existing one.

Order information

Article number 42290000002 Rigidifying Bar Swivel Assy

The unit quantity for this article number is one piece.