

User Instructions

Bathroom Safety Rail L97780

General Safety Warning

Please ensure these instructions are fully read and understood before installation & use. Retain these instructions in a safe place for future reference.

Intended Use

The Bathroom Safety Rail is designed to help steady the user while transferring into or out of the bath. It is not intended to be weight bearing and should not be used by the user to pull up from a sitting position.

Installation Instructions

WARNING: this product may not fit properly on certain unusual or sculptured baths. Inspect for fit and stability before fitting/use.

USE EXTREME CARE when installing the Bathroom Safety Rail onto fibreglass or plastic baths. **DO NOT** over tighten, as clamp may crack the bath.

1. Clean bath wall area where Bathroom Safety Rail is to be applied thoroughly. All contact surfaces must be dry and free of soap residue to ensure secure attachment.
2. Loosen adjustment hand-wheel by rotating anti-clockwise until the platform opening is wide enough to fit around top of bath wall.
3. Place Bathroom Safety Rail onto bath, with hand-wheel facing outside the bath and tighten snugly by turning the hand-wheel clockwise.
4. Test the Safety Rail is secure by pulling backwards, forwards and side to side. If the unit moves, reposition and refit until secure.



Safety Precautions

DO NOT use Bathroom Safety Rail to pull up from a sitting position.

Periodically check the stability and rigidity of the Bathroom Safety Rail.

Cleaning Instructions

Clean regularly, wiping down with a mild detergent or disinfectant solution.

Do not use an abrasive detergent, cleaner or cloth to clean or dry this product.

Manufactured for
NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business Park,
Bardon Hill, Coalville,
Leicestershire.
LE67 1UB UK

Tel. Main Office: 0330 100 00 98

Tel. Customer Services: 0345 121 8111

Email: customerservice@nrs-uk.co.uk

For further details of NRS Healthcare's extensive range of equipment and aids for daily living please visit our website www.healthcarepro.co.uk