

MP10200S

Rollator - Bariatric 8" Wheel

USER MANUAL



Parts of the product



Specification of the product

| Product dimension | Product dimension folded | Handle height adjustment | Weight Capacity | Seat Height | Seat Dimensions | Backrest Height | Wheel diameter |
|-------------------|--------------------------|--------------------------|-----------------|-------------|-----------------|-----------------|----------------|
| 680*738*1020mm | 755*225*900mm | 823-1020mm | 500 LBS | 550mm | 550*380mm | 828mm | 8 inch |

Assembly Instructions

Assemble handles and height adjustment

The rollator is delivered largely pre-assembled remove the lever from the frame, insert the handles into the tube on the frame position of the handles at the desired height.

The handles are correctly adjusted when you are at the same height as your wrists, fix the height by inserting the lever and tighten it, and make sure that both handles are at the same height.



Assemble the shopping basket

Folding the seat forward, hang the hooks of the shopping basket in the frame, and Do not transport too heavy loads (Maximum weight for the basket is 5kg).

Assemble the backrest

The backrest should only be installed when the rollator is in a safe position and the brakes are activated for the backrest, proceed in reverse order.



Parking brakes



To apply the parking brake, press the brake lever down. To release it, apply the brake as usual. Caution! Never apply the parking brake while in motion.

To reduce walking speed, use the lever on the handle

To do this, pull the handle upwards towards you with your fingers.



Folding rollator

To transport or store your rollator to save space, you can fold it up and fold the seat up. Take out the shopping basket and pull up the cross braces, the rollator will fold up.



Unfold rollator

Position your rollator in front of you and apply the brakes.

The rollator unfolds by gently pressing the handles. Press the cross brace down and fold the seat surface down.