

# **USER MANUAL**

Petite Walker 6in Wheels MP10103G



# Parts of the product:



1	Handgrip	6	Rear wheel
2	Hand brake	7	Front wheel
3	Screw	8	Bag
4	Seat	9	Backrest
5	Brake pad	10	Braking cable

## Warranty:

1 year for main frame only, not including any consuming parts( wheels, brakes, and so on...)

#### **Intended purposes**

Rollators empower children to move around more independently, enhancing their confidence and sense of autonomy in navigating their environment. By enabling children to move about more freely, rollators encourage physical activity and participation in daily routines, which is essential for their overall health and well-being. Rollators equipped with storage compartments or accessories such as trays can help children carry items or engage in activities while on the move, promoting greater participation in daily tasks and social interactions.

#### **Intended users**

- 1. Children with Mobility Impairments
- 2. Children with Developmental Delays
- 3. Children Recovering from Surgery or Injury
- 4. Children with Chronic Conditions
- 5. Children with Special Needs

## **Limitation of liability:**

Maxhealth Corporation accepts no liability for damage arising from:

- Non-compliance with the User Manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- The usage of unapproved 3rd party accessories instead of a tested and approved accessories
   Unauthorized modifications and/or use of unsuitable spare parts

A written authorization of Maxx Corporation must be obtained before installing additional adaptations on this wheelchair. Otherwise no liability claims can be made.

#### **Standards and Regulations**

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

Maxhealth Corporation is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

#### Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

### Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Use the rollator only as a walking aid.
- Use the rollator only on an even and solid surface.
- Rollator may not be used as a seat in vehicles, such as a bus or train!
- It is prohibited to stand on the rollator or to transport a person or animal on the rollator.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate
- Use the tray only for small, lightweight objects which are not sharp or hot. Max. load: 5.0 kg
- Use the basket only to transport objects and do not overload it! Max. load: 5.0 kg
- Use on escalators is expressly prohibited!
- The push handles must always be adjusted parallel to the direction of movement – risk of tipping!
- Do not hang any bags on the handle risk of tipping!
  On steep surfaces, use the service brake (pull the brake lever towards you).

## Height adjustment of handles



Insert push handles at least to the first cross hole into the frame tubing. Insert the fixing screw from the inside and tighten using the washer and star screw. To change the height adjustment of the push handles, loosen the fixing screw by turning the star screw counter-clockwise. Then lock the handle at the desired height, as described above.

#### Adjusting the brakes



If the braking is poor or too hard, the brake can be adjusted either at the brake housing above or using the adjusting screw on the Bowden cable below) to achieve the desired braking effect.

The brake is correctly adjusted when the brake lever as well as the brake wires is free of play and the wheels can be rotated without grinding noises when the brake is released.

### **Braking | Parking**



To brake when walking with the rollator, pull both brake handles upwards with your hands. Always brake slowly while applying equal pressure with both hands.

To apply the brakes/park the rollator, press both brake handles downwards until they audibly click into place.

To release the brake, pull both brake handles upwards once again until they are released (brake handles are in the middle position).

The braking effect can be impaired by wear and tear on the tires. For this reason, regularly check the braking effect.



#### Sitting

Place the rollator on an even and solid surface. Then apply the brakes. The rollator must be stable and may not slip. Only sit upright; do not lean backwards Seated position permitted only as shown and against the direction of movement.

# **Folding**

To fold up the rollator, open the folding guard, pull the seat and rollator upwards! Remove accessories, if necessary.

## **Specification Sheet**

Specification Sheet			
Product	PETITE WALKER 6IN WHEELS GREEN		
Product item number	HE2163119G		
Weight capacity	300lbs		
Product dimension L×W×H	560*520 (731-847) mm		
Product dimension folded L×W×H	520*170*760 mm		
Product N.W.	5.5 KG		
Seat dimension L x W	305*305*38mm		
Backrest material	Alum.		
Seat material	PVC		
Front wheel dimension	inch: 6		
Rear wheel dimension	inch: 6		
Frame material	Alum.		
Frame color	Green liquid painting		
Foldable yes/no?	yes		
Detachable footrest yes/no?	no		
Swing-away footrest yes/no?	no		
Height adj. footrest yes/no?	no		
Bag yes/no/optional?	yes		
Cane holder yes/no/optional?	no		
2 in 1 yes/no?	no		
Detachable backrest yes/no?	Yes		
Heel strap yes/no?	no		
Quantity in box (piece):	1PC/CTN		
Product weight including box	6.7KG		
Carton dimension L x W x H	530*180*785 mm		
MOQ(1*40HQ) Q'ty	741 PCS		
FOB port	XINGANGTIANJIN,CHINA		
1			