



**ALLIED  
MEDICAL**  
LIMITED

# USER MANUAL

Adjustable Backrest

MP30101



### Intended Uses

Backrests allow for, or increases independence when using the bed, particularly by compensating for impaired movement or mobility. Aid in providing back support for the user.

### Introduction

This adjustable angle backrest is manufactured from tubular steel with nylon back and removable padded headrest. Adjustable in five positions it can be folded flat when not in user for easy storage and transportation.

### Installation Guide

Place the backrest into the bed with the section of the backrest with the five plastic hook stays facing up. Raise the support section of the backrest which is covered with a blue nylon fabric and with a small head support to the desired height.

**Note:** For safety ensure that the “U” section of the backrest is properly resting within the chosen hook stay once the back support is raised.



### CAUTIONS

- Do not place any obstacles or limbs between the opening sections of the backrest.
- Check regularly that the backrest is properly secure in the plastic hook stay.

### Product information

<b>Product name</b>	BACK SUPPORT
<b>Product number</b>	HE10780020
<b>Product Size</b>	650*750mm
<b>Folded size</b>	600*570*90mm
<b>Adjustable height of backrest</b>	90-560mm
<b>Tube diameter(mm)</b>	Ø22
<b>Frame material</b>	Steel
<b>Frame processing</b>	Powder coating white
<b>Packing quantity(piece)</b>	4PCS/CTN
<b>Carton size</b>	652 x 348 x 618mm
<b>N.W.</b>	12.0KG
<b>G.W.</b>	13.4KG