

USER MANUAL

Chair Raiser Set of 4 SLHA4506



INTRODUCTION

The Chair Risers increase the height of your chair, making it easier for users to get on and off the chair. It fits chair legs up to 63mm in width.

SPECIFICATION

Size (1pc)		Material	Max. Load Weight
145x145x100mm	393g	PP / PU Foam	200kg

INSTRUCTIONS FOR USE

- 1. Place a riser beside each leg.
- 2. Gently lift up the chair and put the leg onto the riser.
- 3. Make sure all the risers are at the same height.

ATTENTION

- The maximum weight capacity of the four risers is 200kg.
- Use on flat surfaces only.
- Not suitable for recliners.
- Avoid uneven weight distribution on the chair.
- Make sure all the legs are properly fitted in the riser before use.
- Periodically inspect for wear and damage.

MAINTENANCE

Wipe clean with a damp cloth.

