

# **USER MANUAL**

## Bed and Chair Raiser Set of 4

SLHA4507



sales@alliedmedical.co.nz | www.alliedmedical.co.nz | 0800 31 61 81

### INTRODUCTION

The Bed & Chair Risers increase the height of your chair or bed, making it easier for users to get on and off beds and chairs.

#### **SPECIFICATION**

Size (1pc)	Weight (4pcs)	Material	Max. Load Weight
140x140x150mm	677g	PP	200kg

#### **INSTRUCTIONS FOR USE**

- 1. Place a riser beside each leg.
- 2. Gently lift up the chair to slide in the riser.
- 3. Make sure all the risers are at the same height.

#### ATTENTION

- The maximum weight capacity of the four risers is 200kg.
- Use on flat surfaces only.
- Not suitable for recliners.
- Avoid uneven weight distribution on the chair.
- Make sure all the legs are properly fitted in the riser before use.
- Periodically inspect for wear and damage.

#### MAINTENANCE

Wipe clean with a damp cloth.

