## Indication / Contraindication

This walking aid is suitable for a single user with limited mobility. It is suitable for indoor and outdoor use.

The purpose of walking devices is to compensate for or at least improve restricted walking ability in users (expansion of range and/or mobility).

Persons with restricted hand/arm functions require special adjustments/versions of handgrips or hand rests/armrests.

For persons with balance disorders are walking frames not recommended, this is to be checked by an experiment in an individual case. if it is necessary a companion must be present.

## Sope of delivery contents

- 1. Rollator
- 2. Basket
- Tray
- 4. Cane Holder
- 5. Operation instructions

## Installation

## Hight adjustment handgrips







Insert the tube of the adjustable handgrip into the frame at least to the "MAX" marking. (Fig. 1)

Fix it in the desired height with the fixing lever by turning it clockwise.

Adjust both handgrips on the same height / level.

Fixing lever itself can be adjusted in the desired position by simultaneously pulling and twisting. For readjustment of the height of the handgrips unlock the fixing lever by turning it counterclockwise. Then fix the handgrip how described above.

**Please note!** The handles must be aligned parallel to each other and following the direction the Rollator is being pushed to. Otherwise the rollator may tip to the side! (Fig. 2 and 3)

## Unfolding



Place the walking device on its wheels.

By pulling one of the pipes of a back wheel you can unfold the frame (1), then press the crossbar down (2) until it easily engages.

Turn the movable folding protection down (3) until this stands vertically to the crossbar.

# Fitting basket and tray



Hook in the basket in the hooks at the frame under the seat plate and place the tray with its small feet in the holes in the seat plate (see picture).


# Adjusting incline of forearm rest



Gradually open the clamp screw on the side until you can move the forearm rest out of the locking position. Once you have adjusted the incline, move the forearm rest back into the locking position, then re-tighten the clamp screw.

## Adjusting the armrest

Loosen the locking lever underneath the armrest by moving it anti-clockwise. Adjust the break lever to the desired position. Then re-tighten the locking lever by turning it clock-wise. (Fig. 7)

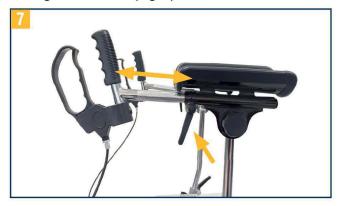

# Breaking / Parking

Pull both brake levers backwards to apply the brakes whilst the rollator is moving (picture 8). Always brake slowly and steady.

To lock the brakes, ensure the Rollator is stationary and press both brake levers forward until they lock into position (picture 9).

To release the brakes from the locked position, pull back on the brake levers. (both brakes are in a middle position).


The effect of the brakes can be affected by abrasion of the wheels. Therefore regularly test the brakes.





# Adjustment of brakes

For desired braking strength adjust the brake either manually at the screw on the housing of the brake (picture 10) or with the suitable tool (not included) on the brake wire (picture 11).

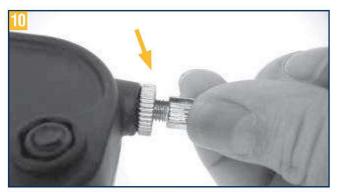



## Sitting

- · Position the rollator on a flat and solid surface.
- Then lock the brakes (parking position).
- The rollator must stand and must not slip.
- Sit straight up and do not lean backwards.

# Folding / Transport / Storage

For folding open folding protection first (picture 13) then pull seat upwards. Walking device can now be transported by trailing or is now suitable for storage.



## Safety instructions

- · Use only as a walking aid!
- · Use only on even and solid surfaces.
- Due to the design, there is an increased risk of tipping sideways! In the outdoor area, pay special attention to the ground! Do not drive any inclines or ramps sideways!
- Always make sure that both forearms are securely and firmly in the armrests!
- · Never use the rollator with one hand!
- The rollator must not be used for standing on or as a wheelchair! It must not be used for transport of persons or animals.
- · Maximum user weight: 120 kg
- The use on escalator's is expressly forbidden!
- Regularly check the screws and fasteners on the rollator and retighten if necessary.
- Use tray only for small, light, but not for pointed or hot objects. Max. load: 5.0 kg
- Use basket only for transport of objects and do not overload. Max. load: 5.0 kg
- Do not expose the product to temperatures above 40°C for a long time, as this can cause the rollator parts to overheat and burns to the user.
- The effect of the brakes can be affected by abrasion of the wheels. Therefore test regulary the brakes.
- When parking the rollator the locking brake should always be engaged to prevent the rollator from unin-

tentional moving.

## Cleaning / Maintenance

Clean regularly by hand with a damp cloth! Do not use any aggressive detergents!

#### Reprocessing

The product is suitable for reprocessing in line with manufacturer's recommendations.