



Squiggles Stander

Spare Parts Catalogue




Supporting
Children

Squiggles Stander Spare Parts - Covers




Pos.	Leckey Sales Code	Description	Min. Order Qty	Colour Options			
				Green	Orange	Blue	Pink
		[Replace XX with colour]					
1	118-XXX	SQST Stander Covers	1	-760	-761	-762	-763
2	118-600	Squiggles Stander Frame	1				
3	118-753-XX	Contoured Headrest Cover	1	-01	-02	-03	-04
3	117-850-05	Flat Headrest Cover - Black Only	1				
3	120-785-05	Flat Headrest Hardware & Upholstery (with laterals) - Black only	1				
3	137-807-05	Head/Hip Lateral Covers - Black only	1				

Squiggles Stander Spare Parts - **Spine Outer**

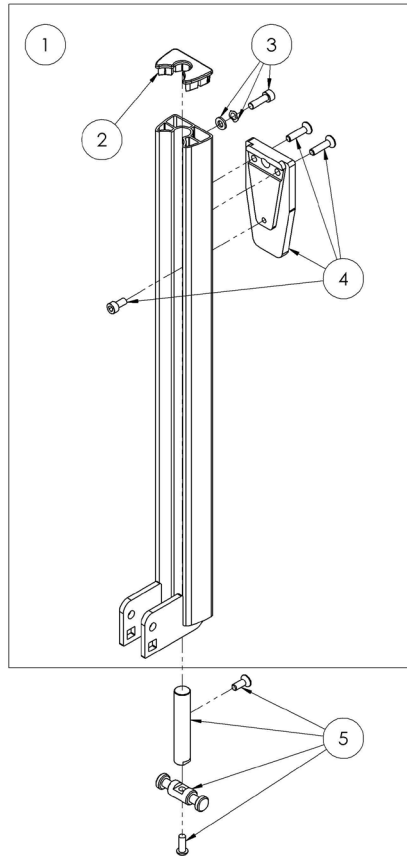


Pos.	Leckey Sales Code	Description	Min. Order Qty
1	118-610	Outer Spine Assembly	1
2	118-154	Extrusion Outer Cap	1
3	118-121	Outer Spine Screw Kit	1
4	118-122	Interface Kit	1
5	118-123	Easel Chassis Base Pin Kit	1

Squiggles Stander Spare Parts - Inner Spine

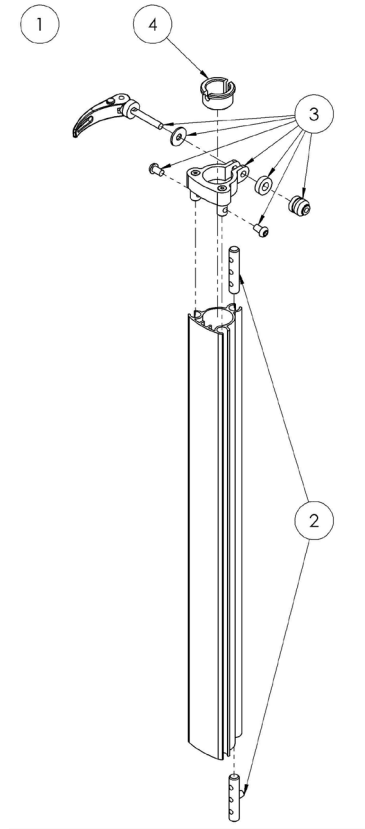


Pos.	Leckey Sales Code	Description	Min. Order Qty
1	118-611	Inner Spine Assembly	1
2	118-124	Inner Extension Kit	1
3	118-712	Cam Lever Assembly	1
4	118-166	Head Support Collar	1

Squiggles Stander Spare Parts - Knee Assembly

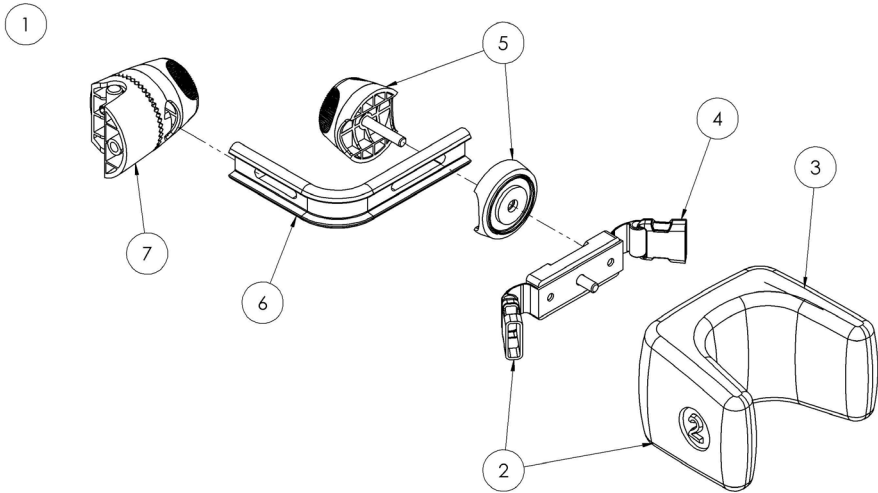


Pos.	Leckey Sales Code	Description	Min. Order Qty
1	118-613	PU Knee Cup Full Assembly [Size 2] [Single]	1
2	118-674	PU Knee Cup & Strap [Size 2] [Single]	1
3	102-1640	PU Knee Cup [Size 1] [Single]	1
3	102-2640	PU Knee Cup [Size 2] [Single]	1
4	118-620	Knee Strap Kit [Single]	1
5	118-673	Spine Orb Assembly [Single]	1
6	118-030	Kneecup Extruded Plate [Single]	1
7	118-675	Knee Orb Assembly [Single]	1

Squiggles Stander Spare Parts - Chest Pad and Orb Assembly

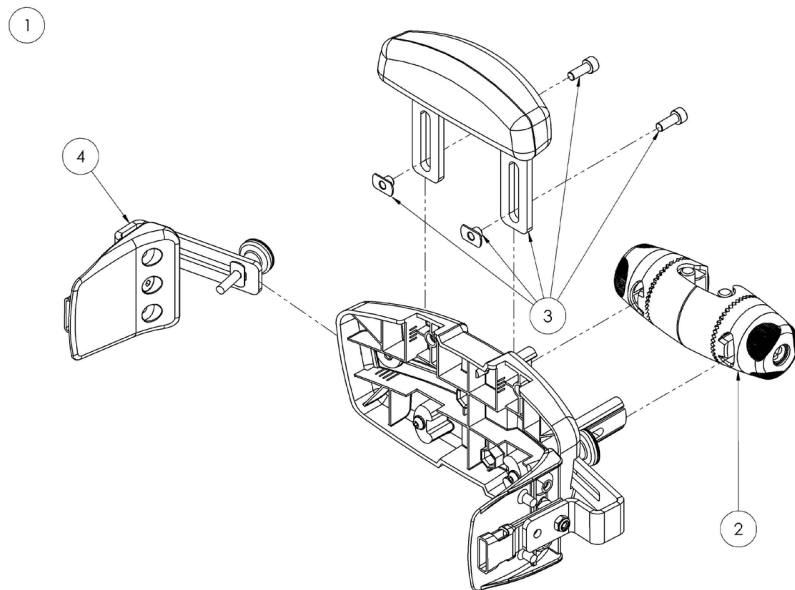


Pos.	Leckey Sales Code	Description	Min. Order Qty
1	118-615	Chest Pad Full Assembly	1
2	118-680	Orb Assembly	1
3	118-664	Sternum Pad	1
4	118-666	Lateral Assembly [Single]	1

Squiggles Stander Spare Parts - Hip Support Pad and Orb Assembly

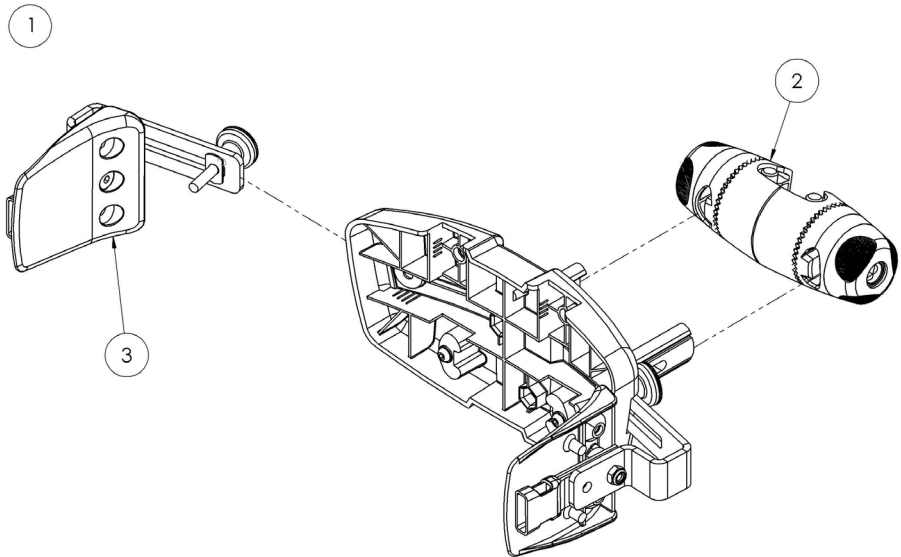


Pos.	Leckey Sales Code	Description	Min. Order Qty
1	118-612	Chest Pad Full Assembly	1
2	118-660	Orb Assembly	1
3	118-666	Lateral Assembly [Single]	1

Squiggles Stander Spare Parts - Footplate

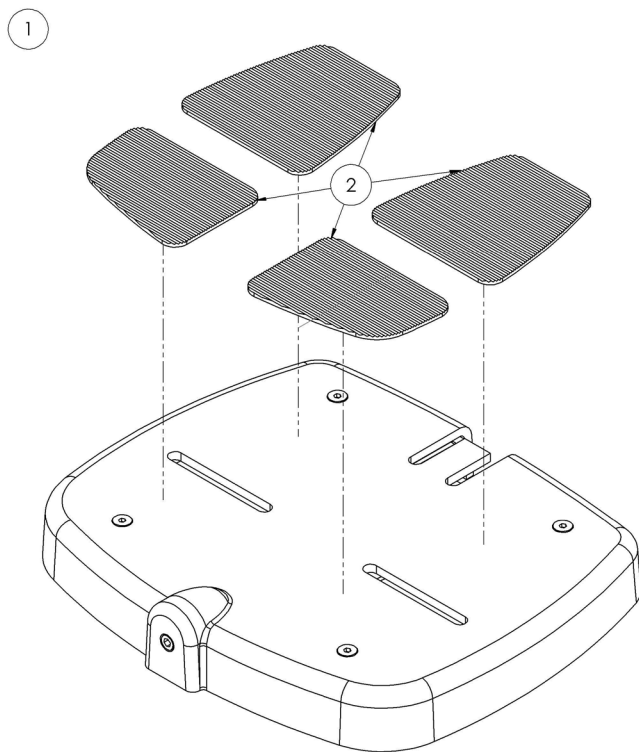


Pos.	Leckey Sales Code	Description	Min. Order Qty
1	118-614	Footrest Assembly	1
2	118-207	Rubber Pad Set	1

Squiggles Stander Spare Parts - Flat Head Support, Cushion & Cover

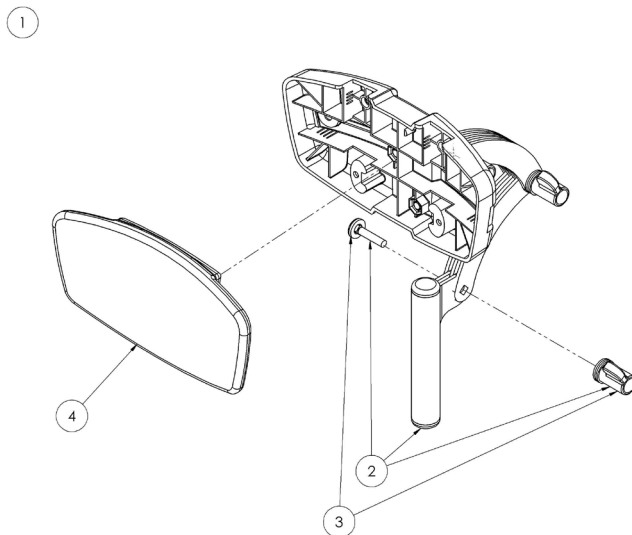


Pos.	Lecley Sales Code	Description	Min. Order Qty	Colour Options			
				Green	Orange	Blue	Pink
		[Replace XX with colour]					
1	117-851-05	Flat Head Support & Cushion - Black	1				
2	117-697	Flat Headrest Support (Hardware Only)	1				
3	137-9056	Wing Nut Assembly	1				
4	117-850-05	Flat Headrest Cover - Black	1				
4	118-753-XX	Contoured Headrest Cover	1	-01	-02	-03	-04

Squiggles Stander Spare Parts - Tray Assembly

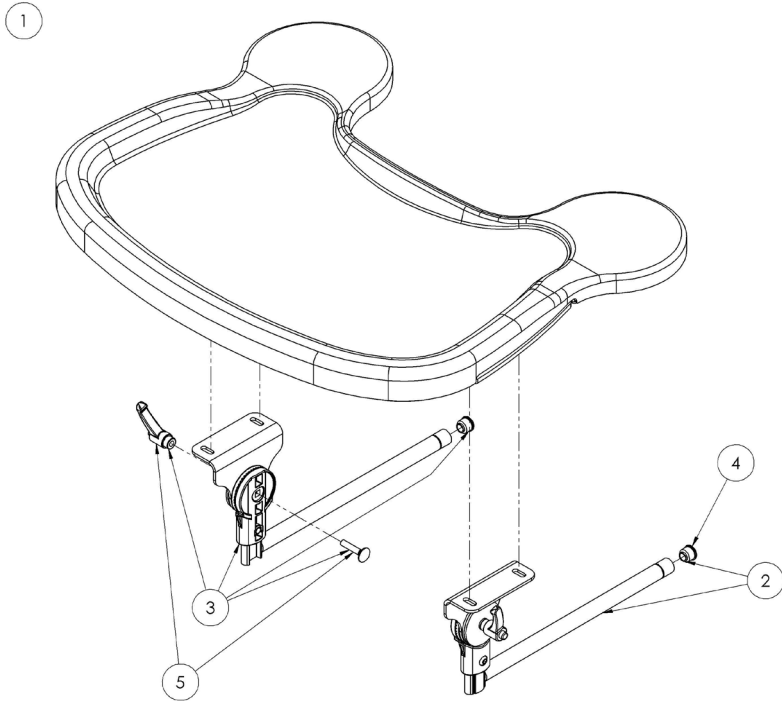


Pos.	Leckey Sales Code	Description	Min. Order Qty
1	118-619	Tray Assembly	1
2	118-710	Tray Depth Adjustment Assembly LH	1
3	118-618	Tray Depth Adjustment Assembly RH	1
4	SF51	Round Tube Insert [Single]	2
5	118-621	Handle Kit [Pair]	1

Squiggles Stander Spare Parts - Tray Support Assembly

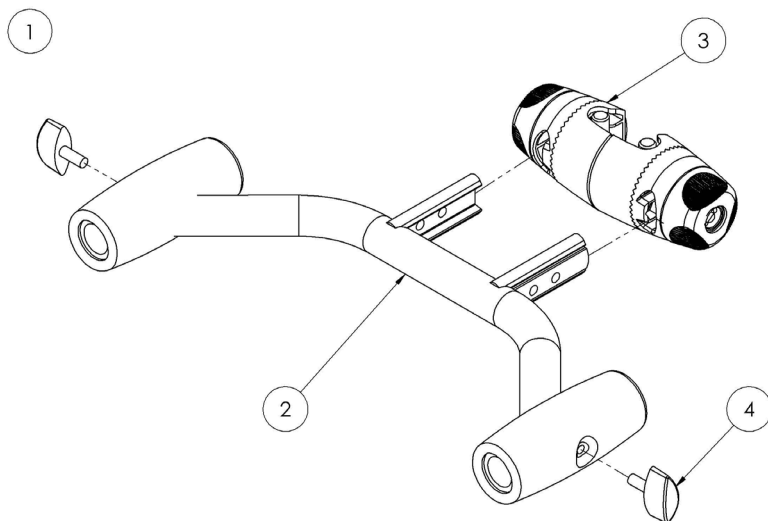

Pos.	Leckey Sales Code	Description	Min. Order Qty
1	118-689	Tray Support Assembly	1
2	118-709	Tray Attachment Assembly	1
3	118-660	Orb Assembly	1
4	SF48	Locking Key M6x16 [Single]	2

Squiggles Stander Spare Parts - Pivot Chassis

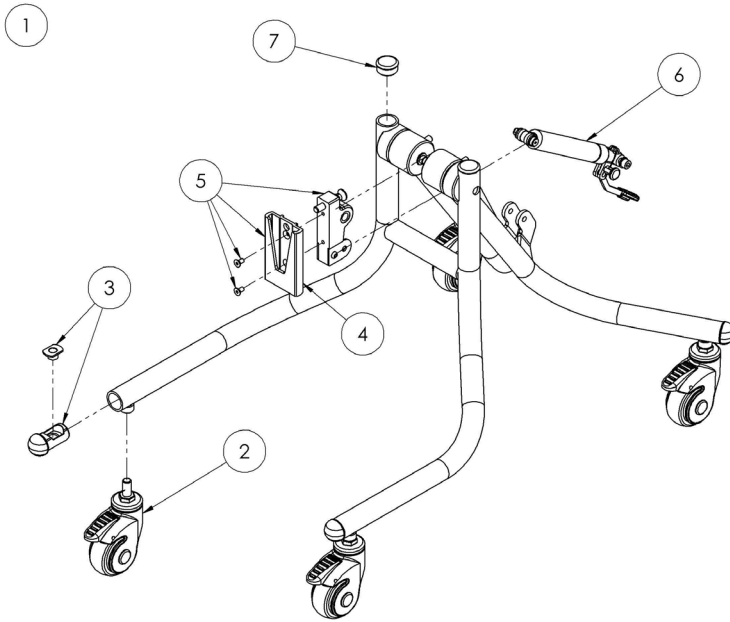

Pos.	Leckey Sales Code	Description	Min. Order Qty
1	118-764	Pivot Chassis	1
2	IA34	Squiggles Castor	1
3	118-161	Castor Tube Insert Kit [Single]	1
4	118-158	Interface Extrusion	1
5	118-713	Pivot Interface Assembly	1
6	118-717	Gas Spring Assembly	1
7	SH62	Tube Insert [4 Pack]	1



Leckey
19c Ballinderry Road
Lisburn BT28 2SA
Northern Ireland

{+44} 28 9260 0750
hello@leckey.com

LS837-01

leckey.com

