
Q-Logic 3 Controller Quick Guide

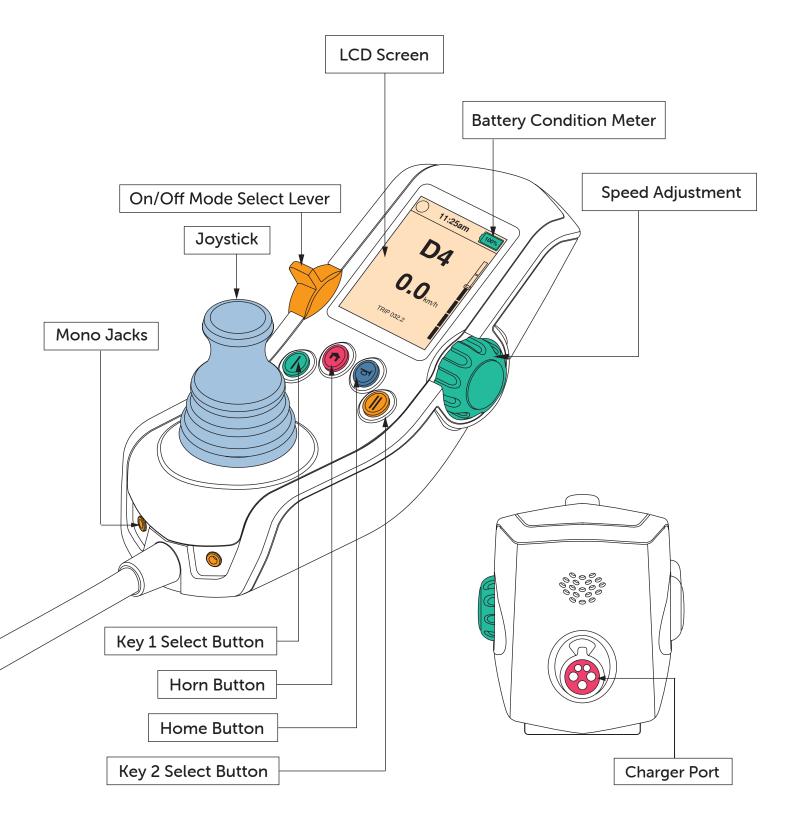



Freephone 0800 31 61 81 Ph 09 415 1685 Fax 09 415 1686

helpis@alliedmedical.co.nz

PO Box 302 250 North Harbour Auckland



A

www.alliedmedical.co.nz

Q-Logic 3 Controller Quick Guide



Ensure you turn off the powerchair for transfers.



Freewheel your powerchair by disengaging both motors.




Never disengage your motors when on a slope!



Locked and in drive mode.


Place unit on level ground and stand to one side when changing between modes.

Unlocked and in freewheel mode.

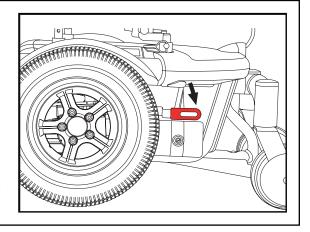



Remember to charge your powerchair every night.

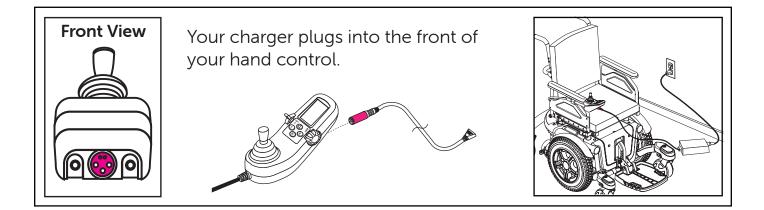



Freephone 0800 31 61 81 Ph 09 415 1685 Fax 09 415 1686

helpis@alliedmedical.co.nz

PO Box 302 250 North Harbour Auckland www.alliedmedical.co.nz

