



GoTo

.....
PRESSURE RELIEF CUSHION

The new **GoTo Pressure Relief Cushion** is available in 2 sizes to pair with each size of GoTo Seat.



1

167-101

2

167-201





Who is it for?

- Current and new customers can benefit from the Pressure Relief Cushion.
- Existing GoTo Seat users can purchase the cushion as an Accessory. This enables children to be involved in the fun and to work on their sitting posture for longer.
- New GoTo Seat customers can purchase the Cushion with their GoTo. Proven to reduce pressure points, the cushion will allow the GoTo to benefit a wider range of children.





What does it do?

The detachable roll can be folded underneath the cushion for added comfort and to **enhance pelvic stability**. Useful to stop bottoms from sliding forward.

What does it do?

Placing the roll under a child's knees can make **long sitting more comfortable**, as this helps create a natural bend in the knee. This cushion position is useful for children with fluctuating muscle tone.





What does it do?



The roll can also be **placed under a child's ankles** if the seat base allows.





What does it do?

.....

The cushion can be used with or without the roll on a **dining room chair**. If suitable for the child, the roll can provide extra comfort and can minimise any discomfort or friction against the chair.



Washing

- Cushion and roll covers are machine washable at **40°C**.
- The inner foam is also machine washable at a **40°C express wash**.
- Please allow both covers and foam to dry fully, do not tumble dry.



Thank You!

