





Figure 4.46

Figure 4.47

Figure 4.48

- 4.11.3 Pelvic belt Fitting
 - 4.11.3.1 Loosen the screw on the back bracket..
 - 4.11.3.2 Fit the pelvic belt. Use the screw you removed and retighten.(Figure 4.48)
- 4.11.4 Maintenance

The pelvic belt is made from 100% polyester fibers and the clasp is made from plastic. It does not require any maintenance but should be inspected every month to ensure that the edges are not frayed and weekly to make sure that it correctly fits the user.

 Maintenance: There is no specific maintenance for the pelvic belt; however, please check if the edge of the pelvic belt is worn or torn before you use it. Meanwhile, please adjust the pelvic belt properly to fit the user. If the pelvic belt is damaged, please replace a new one immediately.

4.12 Manual Parking Brake

This wheelchair is equipped with two manual parking brakes.

4.12.1 They are applied directly against the tires. Push the handle of the

manual brake forward to engage and pull the handle of to release.

- 4.12.2 Braking could be less efficient due to:
 - 4.12.2.1 Worn tire tread.
 - 4. 12.2.2 Low tire pressure. (if equipped with pneumatic tires)
 - 4. 12.2.3 Wet tires
 - 4. 12.2.4 Improper wheel locks adjustment.
- 4.12.3 The manual parking brakes are not designed to be used for a moving