


E10 Push Chair

User Manual

Model# Series
63000000
UMEI006-16LIT



 **WARNING: AVOID SERIOUS INJURY FROM FALLING OR SLIDING OUT. ALWAYS USE THE SEAT HARNESS. NEVER LEAVE CHILD UNATTENDED. IMPORTANT: KEEP THESE INSTRUCTIONS FOR FUTURE REFERENCE. A CHILD'S SAFETY IS YOUR RESPONSIBILITY. THIS PUSH CHAIR IS SUITABLE FOR ONE CHILD.**

GENERAL INFORMATION & SAFETY:

- Follow all instructions carefully. Incorrect use will cause damage to push chair.
- This product has been designed to take the maximum weight of 90lbs (40kg).
- Children should be harnessed in at all times and should never be left unattended.
- **IMPORTANT - It is important for your child's safety that the harness be correctly fitted and adjusted. If not adjusted correctly in accordance with the instructions, the stability of the push chair will be compromised.**
- The harness and seat belt are not a substitute for proper adult supervision.
- The child should be clear of moving parts while making adjustments.
- **IMPORTANT - DANGER: It is important that care is taken to ensure that children are kept clear of the push chair when folding or unfolding. Pinch points and scissoring actions are unavoidable during these operations.**
- Do not carry additional children, goods or accessories in or on this push chair except as permitted in this manual. They may cause the push chair to become unstable or place unneeded stress on the push chair that could lead to breakage.
- Overloading, incorrect folding and the use of accessories, e.g. child seats, bag hooks, rain covers, buggy boards, etc., other than those approved by the manufacturer may damage or break this push chair.
- Any damage caused by the use of accessories not supplied by manufacturer will not be covered by the terms of our warranty.
- Never leave child in the push chair when ascending or descending stairs or escalators or when travelling on other forms of transport.
- The push chair should always open and fold easily. Do not force it. It may be necessary to re-read the instructions.
- Negotiate curbs and rough ground carefully. Repeated impact will cause damage to the push chair.
- **Never allow children to stand on the footrest.**
- Do not leave the push chair exposed to heat, i.e., by a radiator or in direct sunlight.
- Do not place items on the top of the hood as it may cause damage to the canopy.

MAINTENANCE AND REPAIR:

This push chair requires regular maintenance by the user. Regularly check all rivets and connecting devices for tightness and security. Inspect all brakes, wheels and tires and replace or repair if necessary. Check all safety devices for correct operation, in particular the primary and secondary locks. These should be free to move at all times. Never continue to use a product that is not structurally sound.

- We recommend a service every 12 months.
- If wheels squeak, lubricate sparingly with Teflon or silicone coating. Do NOT use oil or grease based products, as this will attract dirt, which will clog the movement.
- Only manufacturer replacement parts should be used. It may be unsafe to use parts not supplied by manufacturer.

CLEANING:

- Clean frame with a damp cloth and a mild detergent and dry thoroughly.
- If the push chair chassis parts have been exposed to salt water, we recommend that they be rinsed down with fresh (tap) water as soon as possible.
- The canopy may be sponged lightly using a damp cloth and a mild detergent. The seat cover, head cushion, lateral pads and harness pads may be washed. Line dry thoroughly before reuse. Gray Harness can be placed in a mesh bag and machine washed on gentle cycle only.
- Do not fold or store the product while wet and never store in a damp environment, This can cause mildew to form.

The manufacturer reserves the right to make design changes to any of their products as part of their continuous design improvement program

IMPORTANT:

- **READ THESE INSTRUCTIONS CAREFULLY BEFORE USE AND KEEP THEM FOR FUTURE REFERENCE.**
- **APPLY BRAKE WHEN PARKING THE PUSH CHAIR.**
- **YOUR CHILDREN'S SAFETY MAY BE AFFECTED IF YOU DO NOT FOLLOW THESE INSTRUCTIONS.**

PARTS:



REAR AXLE



FOOTWELL



FRONT
WHEELS



REAR
WHEELS



CANOPY



LATERAL
PADS



SEAT FABRIC



FRAME &
SHOPPING
POCKET



FOOT BLOCK




**NOTE: THIS PART
IS JUST PACKING
MATERIAL**

1. FRAME

Open frame in the upright position.

When primary & secondary lock is engaged, an audible CLICK will be heard.

NOTE: Shopping pocket rigid inserts can be removed from shopping basket.

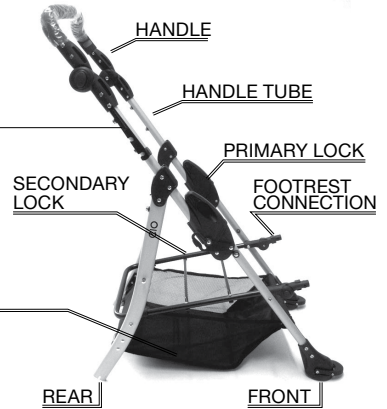

2. REAR AXLE

Push rear frame legs into rear axle.

When lock is engaged, an audible CLICK will be heard.



3. REAR WHEELS

Push rear wheel onto left rear axle. Repeat for right.

When lock is engaged, an audible CLICK will be heard.



4. FRONT WHEELS

Push front left wheel up into front frame leg. Repeat for right.

Lubricate sparingly with Teflon or silicone coating.

When lock is engaged, an audible CLICK will be heard.



5. FOOTREST

Orient footrest so that it is facing up. Push right and left footrest tubes into the frame at the same time.

When lock is engaged, an audible CLICK will be heard.



6. SEAT FABRIC

Double button on the handle tube (right & left sides).

When button is engaged, an audible SNAP will be heard.



6A. SEAT FABRIC

Button support straps to each rear leg.

When button is engaged, an audible SNAP will be heard.



6B. UNDER SEAT FRAME STRAPS (x2)


FRONT STRAP
(above bolt, below
footwell adjustment)

REAR STRAP



6C. IMPORTANT: UNDER SEAT STRAP D-RINGS


Thread up through the bottom, then down through the top.

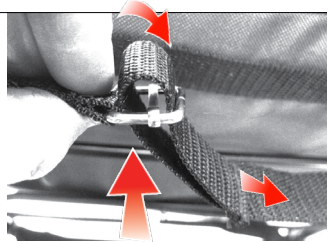

6D. UNDER SEAT STRAP D-RINGS

Loose tail of under seat strap should be positioned between the taught strap and the seat bottom.


DO NOT overtighten.

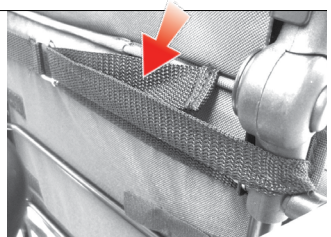

6E. FOOTWELL SNAPS (x2)

Button footwell fabric to footwell snaps.

When button is engaged, an audible SNAP will be heard.

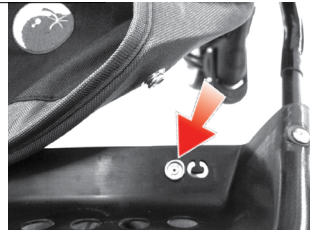

6F. FOOTWELL VELCRO (x2)

Wrap velcro around footwell frame and fasten.

Repeat for right.



6G. SIDE SADDLE BAGS

Fold over primary frame locks and velcro down.

Repeat for right.




7. CANOPY

- A** Slide canopy clip down into frame connector. DO NOT force. Be sure clip is clear of fabric obstruction. Repeat for left.

When lock is engaged, an audible CLICK will be heard.


- B** To remove canopy, press down clip then slide upward.

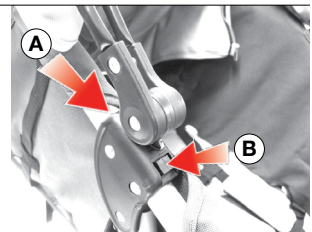

CANOPY SNAP & ZIPPER


Fasten stabilization snaps and zipper the canopy sides. Zipper and snap should be positioned just below the handle adjustment button. Repeat for left.

NOTE: You can remove the rear half of the canopy by unzipping the center seam.

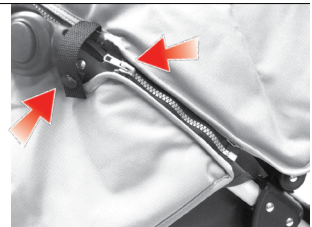

8. PADDED LATERALS (x4)

Feed both velcro straps through slots located at the edges of seat & back surface.

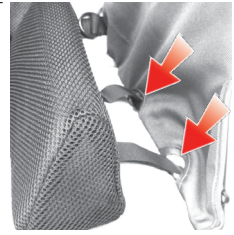

LATERAL VELCRO STRAPS

Pull velcro straps through to the outside of the seating (or back) surface. Then wrap around and fasten.

Repeat for all straps.



9. SEAT BELT / SHOULDER HARNESS / FOOT BLOCK OPERATION





WARNING: Avoid serious injury from falling or sliding out.

Always use the crotch strap in combination with the waist belt and shoulder straps.


9.A HEIGHT-ADJUSTABLE SHOULDER STRAPS

Unclip chest pad clip then feed straps down through backrest loops to adjust the shoulder strap anchor point. Reconnect chest pad clip.

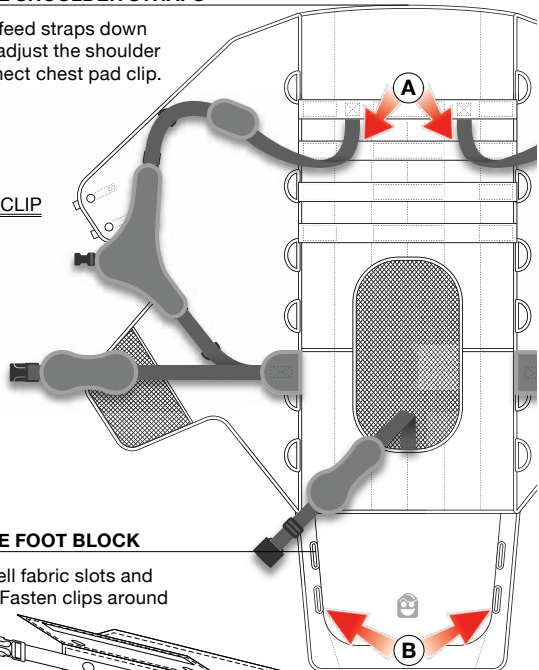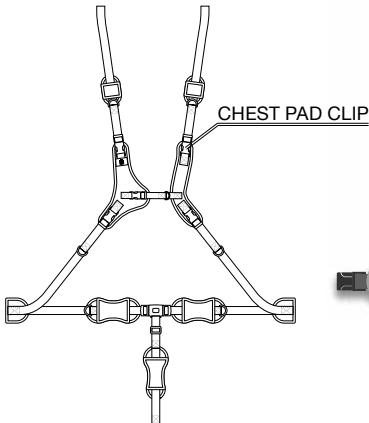

9.B HEIGHT-ADJUSTABLE FOOT BLOCK

Feed straps through footwell fabric slots and around the footwell frame. Fasten clips around the back side and tighten.

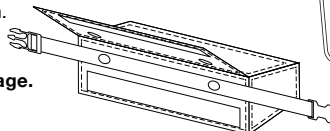

See installed on next page.

9.B HEIGHT-ADJUSTABLE FOOT BLOCK (cont.)

Foot block shown installed.

NOTE: Block can be adjusted higher or lower.

Feed strap around outside of frame.



11. PARKING BREAK

Using foot, push up to disengage and press down to engage.



12. RECLINE STRAP

UP: Push up on recline buckle and pull down on strap.

DOWN: Pinch arrow button on recline buckle and pull down.



13. FOOTWELL ANGLE

To adjust footwell angle, press the button on both sides simultaneously.

When lock is engaged, an audible CLICK will be heard.



14. FRONT WHEEL SWIVEL LOCKS

To engage front wheel swivel locks, press tab up to unlock and press tab down to engage. Repeat for left.

When lock is engaged, an audible CLICK will be heard.

NOTE: For lock to engage, front wheels must be facing forward.



15. FOLDING


- A** Fully recline push chair.
- B** Disengage rear canopy hood from seat back.
- C** Place handle in upright position.
- D** Unlock front swivel wheels (both sides facing forward) .
- E** Standing behind the push chair, loosen safety lock on right side of stroller then pull up to release left and right frame locks.



- F** Frame locks will disengage.
- G** Push down on handle of EIO.




- H** Engage closure clip.

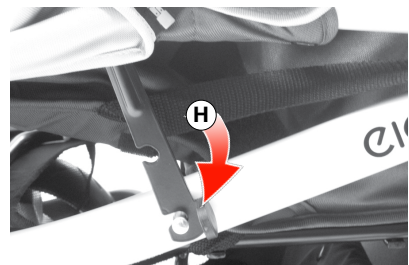

FABRIC CLEANING:

- You can machine wash the EIO seat fabric in cold water with a mild detergent. To minimize shrinkage, simply reattach the seat fabric to the frame to dry. (DO NOT put seat in the dryer)
- Be sure to unzip and remove all three rigid boards from the seat before washing.

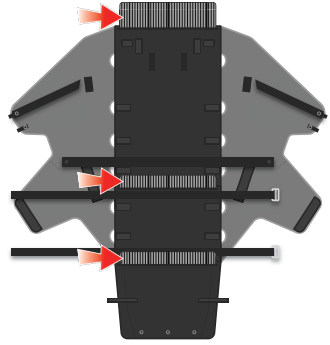

LIMITED WARRANTY:

Your push chair is guaranteed to be free from any manufacturing defects for a period of 2 years from the date of purchase under normal use and provided it is used in compliance with the operating instructions. This warranty extends only to the original retail purchaser and is only valid when supplied with proof of purchase. Please retain proof of purchase with this Limited Warranty.

Should a repair be needed please contact the retailer from whom the product was purchased.

Please note that the warranty will not be applicable if:

- A fault is the cause of misuse or poor maintenance.
- Repairs are carried out by a third party.
- The push chair is faulty due to general wear and tear resulting from everyday use.
- A push chair is damaged as a result of an accident.
- The manufacturer is not satisfied that the stated warranty terms and conditions have been met.

AUTHORISED EUROPEAN REPRESENTATIVE

GLOBAL CAREHAB
ROSENVANGS ALLE 32
8260 VIBY J
DENMARK
+45 24 65 55 50



T: 1 315 429 8407
F: 1 315 429 8862

© 2016 Bergeron By Design

The information contained herein is subject to change without notice.
Bergeron By Design shall not be liable for technical errors or omissions herein.

⚠ WARNING: Prevent serious injury or death from falls or sliding out

- Always secure child in the restraint
- Never leave child unattended
- Always keep child in view
- Max User Weight 90lbs or 40.82kg



Bergeron By Design
15 South Second St.
Dolgeville, NY 13329
1.866.529.8407

two year
warranty