J623 2.0 with Lift and Tilt Quick Reference Guide





The VR2 Controller has 6 Buttons:

- o On / Off
- o Horn
- Left speed button (slow down)
- Right speed button (speed up)
- o Tilt
- o Lift



TILT

- Press tilt button, light at top of button will switch on.
- You are now in tilt mode. Note: Powerchair will not drive in this mode.
- Pull back on the controller to tilt the seat back and push forward to tilt seat down.
- o Push tilt button again, light will switch off.
- o Powerchair will now drive again.



LIFT

- Press lift button, light at top of button will switch on.
- You are now in lift mode. Note: Powerchair will not drive in this mode.
- Pull back on controller to lift the seat up and push forward to lower seat down.
- o Push lift button again, light will switch off.
- o Powerchair will now drive again.